

# Xwhite® **VISIBLE** VISIBLY WHITER TEETH

Xwhite® VISIBLE - professional dissolving whitening strips have been developed to provide convenient, no-fuss teeth whitening, with a simple application method and no strip to remove afterwards.

**Remove stains from coffee, wine and smoking**  
**Bleaching in motion (in office, car, on the way)**  
**Premium strength formula, enamel-safe dentist approved formula.**


Our enamel-safe Xwhite® Visible dissolving whitening strips enable you to safely and comfortably achieve whiter teeth by applying the strips for 14 days, 20 minutes twice a day. Also can be used 28 days 1x daily (monthly supply)



## DIRECTIONS

Use following the simple instructions below:

1. Moisten teeth with tongue. Apply one strip (either side) to lower teeth, avoiding gumline.
2. Fold excess strip behind teeth. Strip will adhere to teeth. Repeat for upper teeth.
3. Allow to dissolve. The strips should be fully dissolved after approximately 20 minutes.

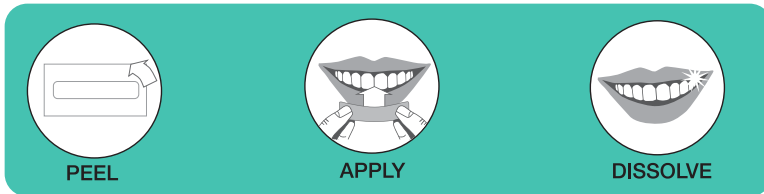

## FREQUENTLY ASKED QUESTIONS

**Q: What is the active ingredient in the whitening strips and is it safe?**

A: The active whitening ingredient is hydrogen peroxide which is a safe tooth whitening ingredient that is used throughout the world by dentists.

**Q: Will the strips affect my dental work?**

A: The whitening strips will ONLY whiten natural teeth; they will not affect caps, crowns, veneers, bridges, fillings or dentures. Do not use on loose teeth, temporary fillings or orthodontic appliances. Consult your dentist if you have any dental concerns.

**Q: When will I see results?**

A: Typically after 8 to 12 applications, although some people will notice a difference sooner. For best results complete the entire course of treatment. You may choose to do multiple courses of treatment depending on how white you want your teeth to be.

**Q: Can the whitening strips be in contact with my gums?**

A: Yes, but such contact should be minimised to avoid gum irritation.

**Q: Should I brush my teeth before / after use?**

A: Do not brush your teeth before or immediately after use strips. If you wash your teeth, wait at least 30 minutes. After whitening, rinse your mouth with water to remove any leftover whitening gel.

**Q: White spots sometimes appear on my teeth after whitening, should I be concerned?**

A: No. These white spots are temporary; they are not harmful and will disappear within 24 hours.

**Q: Can I swallow saliva when using the whitening strips?**

A: Yes, you can. It is perfectly safe to swallow saliva when using whitening strips.

**Q: What if I experience tooth sensitivity or gum irritation?**

A: It is not uncommon to experience tooth sensitivity and / or gum irritation whilst whitening your teeth. This is temporary and NOT harmful. To alleviate symptoms reduce the frequency of treatments or cease treatment until the sensitivity / irritation has subsided. If the problem persists, discontinue use and consult your dentist.

## MAINTAINING YOUR WHITE SMILE

As with all whitening treatments you will need to repeat the whitening process in the future to maintain your white smile. Repeat courses can be done every 12 months. Alternatively, you can use 4-6 strips on a monthly basis to keep topping up your white smile.

## INGREDIENTS

Pectin, PVP, Glycerin, Hydrogen Peroxide, Aqua, Aroma, (Peppermint Flavour), Polysorbate 80, Sucralose

For batch number & best before date refer to outer packaging.

For all information please visit [www.xwhitesmile.com](http://www.xwhitesmile.com) or write to us at [hello@xwhitesmile.com](mailto:hello@xwhitesmile.com).