

Xwhite® ADVANCED WHITENING PEROXIDE FREE TECHNOLOGY

Professional teeth whitening strips

Xwhite® ADVANCED WHITENING - teeth whitening strips provides peroxide –free technology for safe & effective whitening. Patented formula enables beautiful white smile after just 14 days.

Contains Pthalimidoperoxycaproic acid (PAP) in combination with P³Technology. PAP is an effective whitening agent and an alternative to hydrogen peroxide. When in contact with the saliva in the mouth it breaks down to produce “active oxygen” which in turn dissolves the large molecules that cause the staining on the tooth. The innovative combination of the PAP and P³Technology gives a whitening effect that is the equivalent of a 6% hydrogen peroxide formula. The formulation has been shown to be completely safe for tooth enamel.

- **56 strips**
- **Peroxide free, sodium chlorite free**
- **Remove years of stains (coffee, dark drinks and smoking)**
- **Easy use, no slip strip, fresh mint flavour, whitens up to 5 shades**
- **Last 6+ months**

Our enamel-safe Xwhite® ADVANCED WHITENING -teeth whitening strips enable you to safely and comfortably achieve whiter teeth by applying the strips for 14 days, 40 minutes per day.



Designed and formulated by UK dentists.

DIRECTIONS

Use following the simple instructions below:

1. Moisten teeth with tongue. Apply one strip (either side) to lower teeth, avoiding gumline.
2. Fold excess strip behind teeth. Strip will adhere to teeth. Repeat for upper teeth.
3. Allow to dissolve. The strips should be fully dissolved after approximately 20 minutes.
4. Repeat again. (Use 2 daily but one after the other)



FREQUENTLY ASKED QUESTIONS

Q: What is the active ingredient in the whitening strips and is it safe?

A: The active whitening ingredient is phthalimido-peroxy-caproic acid (non-peroxide) which is a safe tooth whitening ingredient which has a significant benefit on whitening.

Q: Will the strips affect my dental work?

A: The whitening strips will ONLY whiten natural teeth; they will not affect caps, crowns, veneers, bridges, fillings or dentures. Do not use on loose teeth, temporary fillings or orthodontic appliances. Consult your dentist if you have any dental concerns.

Q: When will I see results?

A: They immediately start working to remove common stains like coffee, fruit juice and red wine. For best results complete the entire course of treatment.

Q: Can the whitening strips be in contact with my gums?

A: Yes, but such contact should be minimised to avoid gum irritation.

Q: Should I brush my teeth before / after use?

A: Do not brush your teeth before or immediately after use strips. If you wash your teeth, wait at least 30 minutes. After whitening, rinse your mouth with water to remove any leftover whitening gel.

Q: White spots sometimes appear on my teeth after whitening, should I be concerned?

A: No. These white spots are temporary; they are not harmful and will disappear within 24 hours.

Q: Can I swallow saliva when using the whitening strips?

A: Yes, you can. It is perfectly safe to swallow saliva when using whitening strips.

Q: What if I experience tooth sensitivity or gum irritation?

A: There is no tooth sensitivity and/ or gum irritation whilst whitening your teeth.

MAINTAINING YOUR WHITE SMILE

As with all whitening treatments you will need to repeat the whitening process in the future to maintain your white smile. Repeat courses can be done every 6+ months. Alternatively, you can use 6-12 strips on a monthly basis to keep topping up your white smile.

INGREDIENTS

PVP, Hydroxypropylcellulose, Cyclodextrin, Aqua (water), Cellulose gum, Aroma (flavour), Glycerine, Sodium hexametaphosphate, Pentasodium triphosphate, Polysorbate 80, Phosphoric acid, Phthalimido-peroxy-caproic acid, Potassium sorbate, Sucralose, BHT, Limonene, Linalool, Citronellol

For batch number & best before date refer to outer packaging.

For all information please contact us at 01/5620-325, visit www.whitesmile.com or write to us at info@xwhite.hr.